

WOUNDSEAL[®]

TOPICAL POWDER

For Minor External Bleeding

STOPS BLEEDING INSTANTLY!

POWDER + PRESSURE = INSTANT SCAB

DIRECTIONS FOR USE: Clean wound, apply powder only when bleeding resumes (powder needs blood to work).

- 1 TWIST** to remove cap. Puncture foil completely with end of cap.
- 2 POUR** onto bleeding wound, completely covering wound.
(To ensure optimal flow of powder, don't squeeze tube while pouring.)
- 3 PRESS** for 30 seconds to form a seal/scab and stop bleeding.

Repeat application if needed. The protective scab is safe to leave in place until it falls off naturally. Occasionally upon application a temporary warming or stinging sensation may be felt.

WARNINGS: For external use only. Not intended to replace stitches. Do not apply powder to visibly infected wounds. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Keep out of reach of children. Seek medical attention if bleeding persists.

CAUTIONS: Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged, opened, or if powder has clumped and does not flow freely. Discard contents after use since powder is no longer sterile. Powder is guaranteed to work only if it has not been exposed to air or humidity.

One WoundSeal tube is sufficient to cover a wound area of up to 3.9 sq. cm (.6 sq. in.) to a depth of 2mm.

Please visit www.woundseal.com or call 1-800-722-7559 to get more information and/or share your comments.

Ingredients: Hydrophilic polymer and potassium ferrate.

See insert for complete bilingual usage instructions before applying.

Biolife LLC, Sarasota, FL 34243. Patents 6,521,265 and 6,187,347. Other U.S. and international patents pending.

LOT



51708
2020-05-31



Made in the USA

STERILE R


CSC029

REF

CSU004



8 34061 00004 0

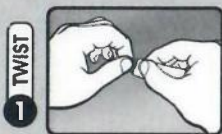

A LEADING BRAND: HOSPITALS USE



CLINICAL
ADVANCES[®]

BLOODSTOP[®]
HELPS STOP BLEEDING QUICKLY

CURAD[®] BLOODSTOP[®] is a water soluble gauze which stops bleeding in less than a minute. When in contact with blood, the gauze turns into a clear gel that expands, adheres, and creates gentle pressure to seal the wound.

How to use:

- 1) Clean the wound.
- 2) Apply BLOODSTOP on the bleeding wound. Can be cut to size.
- 3) Apply light pressure until the bleeding stops completely, usually less than a minute.
- 4) If desired the BLOODSTOP material may be moistened around the wound so it adheres to the skin.
- 5) Once bleeding has stopped rinse off excess BLOODSTOP by saturating with clean water or saline and gently wiping away the gel. Be careful not to disturb the wound itself.
- 6) If bleeding has not stopped after the initial application of BLOODSTOP you may reapply following the previous instructions. If bleeding continues contact a physician.



How to use for nosebleeds:

- 1) Fold BLOODSTOP into a small strip.
- 2) Pack the BLOODSTOP into the nostril as close to the source of bleeding as possible.
- 3) Compress the bleeding nostril to assist in stopping blood flow.
- 4) Once the bleeding has stopped, gently pull or squeeze the gauze gel out of the nostril. Gently rinse away the remaining gel with clean water.
- 5) If bleeding has not stopped after the first application of BLOODSTOP you may reapply more following the previous instructions. If bleeding continues contact a physician.

